


MANUAL
LIFTING CHAIR



Model Number: ILG-325

**PLEASE READ ALL INSTRUCTIONS
BEFORE USING THE ITEM !**

ASSEMBLING THE CHAIR

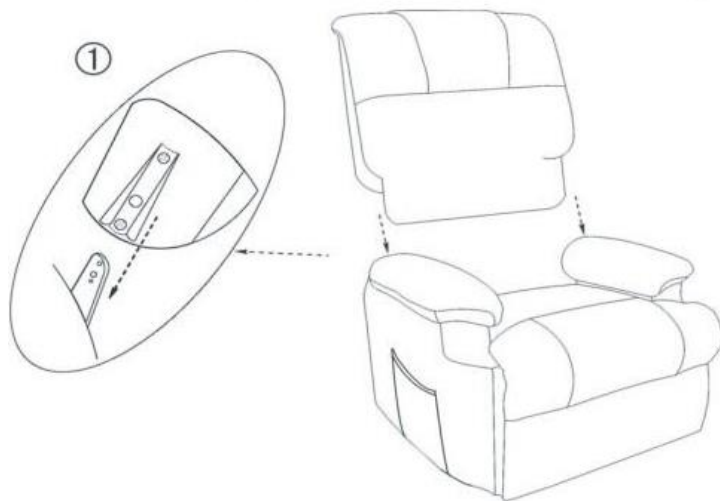

CONNECT THE WIRES

Connect the transformer to the chair



Connect remote control to the chair.




Remark:

1. The motor extension cord is attached on the rear rail at the bottom of the seat.
2. The remote control of the actuator (lifting and reclining) has been connected.

REMOTE CONTROL

Actuator (lifting and reclining)

- Press and hold the **Extended** button to raise the chair to full-lift position.

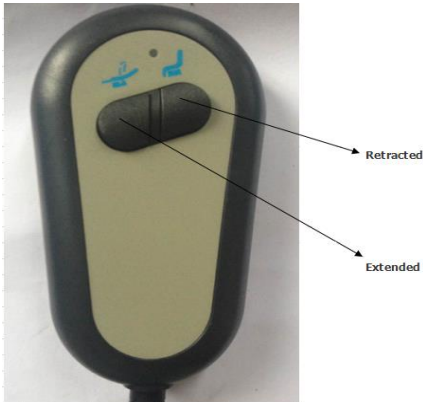

- Press and hold the **Retract** button to lower the chair to seated position.
- Press and hold the **Retract** button to raise the legrest and recline the backrest.
- Press and hold the **Extended** button to return to seated position from reclined position.

IMPORTANT SAFETY INSTRUCTIONS

When using an electrical appliance, basic precautions should always be followed, including the following,

- Never sit or lean on the remote control and never place heavy objects on the remote control.
- Do not put the remote control on floor.
- Do not drag the cables forcefully.
- Never use the remote control with wet hands.
- Do not touch the buttons with sharp objects.
- Do not unplug the remote control without turning it off before.
- This product is for indoor use only.
- Always unplug the chair from the power supply after use and before cleaning.
- Close supervision is necessary when this product is used by on or near children, or disabled persons.
- Use this product only for its intended use as described in this manual.
- Do not use attachments not provided with this unit.

Health cautions

1. If you have any concerns regarding your health, consult your doctor before using this product.
2. If you experience pain in muscle or joint for a prolonged period of time, discontinue using and consult your doctor.

Persistent pain could be a symptom of a more serious condition.

Care and Maintenance

1. To store

- Keep the unit in a clean and dry condition.
- For long time storage, cover the chair up to protect from dust..
- Keep the unit away from direct sunlight or high temperature.

2. To clean

- Make sure the unit is switched off and unplugged from power before cleaning.
- Clean the cover by wiping down with a damp cloth and using a mild detergent or suds. Pat dry with clean and dry cloth.
- Do not use abrasive cleaners, brushes, gasoline, kerosene, glass/furniture polish, paint thinner or water hotter than 120F to clean the unit.
- Use a clean and dry cloth to clean the remote control.

iLIVING USA

239 Harbor Way

South San Francisco, CA 94080

Toll: 800-317-1688 or 650-952-2257

Email: sales@ilivingusa.com